

Touch Screen Help Menu

1: Single-tap stimulus Current or pulse Duration to focus and adjust with +/- buttons.

2: Press to display QRS Detect Menu

3: Increment / decrement of Sx

4: Sets S1 to displayed % of Average RR interval

5: One Touch Overdrive Pace at displayed S1 and Channel. Slide right/Left to raise/lower S1

6: Large Increment / Decrement of focused Parameter (e.g. Sx by 100ms)

7: Reverse last auto-decrement

8: Display numeric keypad
9: Toggle pacing On/Off or keep depressed to pace

10: Increment / Decrement of focused Parameter (e.g. Sx by 10ms)

11: Disable Focused Parameter, e.g. Sx

12: Edit Focused Parameter or Enable Sx

18: Exit EPS320 Program to Standby Mode



17: Depress for 1 sec for Emergency Pacing

16: Flip Page A <-> B Temp. Overdrive Pace Temporary R-Synced Sx Anti-Tachycardia Pacing Procedure Menu Help

15: Press for Alternate Menus

14: Enable control by Touch Screen, Keyboard or Both

13: Press for help on Touch Zones

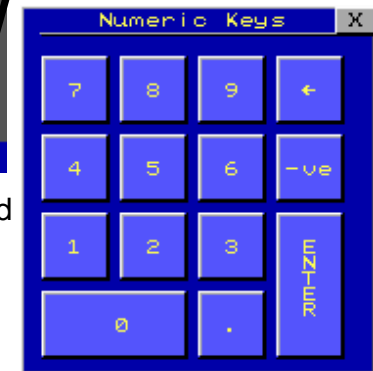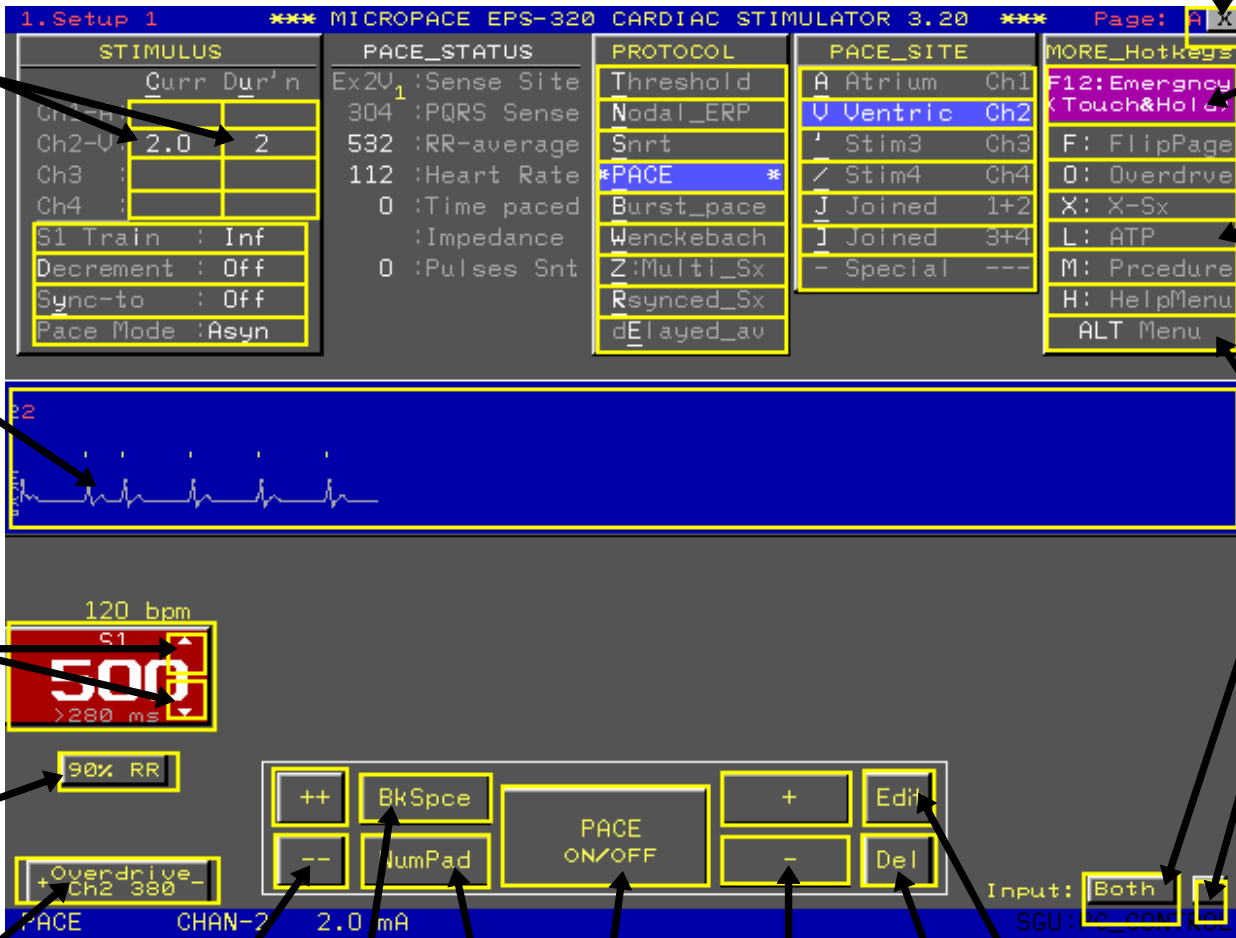

Double touch any numeric value to edit with Numeric keypad