



**Contents and
Instructions for use of the
Touch Screen
MP3113-GE & MP3113-B
For EPS320 Cardiac Stimulator**

Rx Only

This document describes the contents and warnings and precautions for the Micropace Touch Screens MP3113-GE and MP3113-B for use only with Micropace EPS320 series of Cardiac Stimulators. Before using this product, please refer to and read in their entirety (i) User Instruction Manual (MP3395) and (ii) Installation Manual (MP3069 or MP3156) for the EPS320 Cardiac Stimulator. Refer also to the ELO Entuitive Touchmonitor User Guide on the enclosed CD ROM or on the Elo website: <http://www.elotouch.com/support/Documentation/manuals.asp>

WARNINGS AND PRECAUTIONS

Warning:

Installation is to be done by qualified representatives of Micropace Pty Ltd or its authorized distributor.

Warning:

In order to prevent electrocution hazards, always inspect the product prior to use for evidence of damage to the device, associated cables, and connectors; do not use if product appears damaged.

Warning:

Product is not suitable for sterilization.

Warning:

In order to prevent electrocution hazards, ensure that this touch screen and any accessories are powered from a medical grade isolation transformer such as the Micropace MP3107; this includes for example any external mains-powered speakers you might connect to the system.

Precaution:

Protect product and cables from ingress of liquids and body fluids. If soiled refer to cleaning instructions in the EPS320 User Instruction Manual and the ELO User Guide referenced above.

Precaution:

The ELO touch screen serial cable in this box (blue-green connectors) has same connectors as the Micropace Serial Cable MP3033 and MP3033A (DB9 socket to DB9 plug), however is not interchangeable with it; the ELO serial cable may work with the SGU but the MP3033(A) will not work with the touch screen. Ensure that you use the MP3033 / MP3033A to connect the SGU (MP3008) to the Micropace Bona Light computer.

Quick Installation Guide

1. This Touch Screen is for new installations only. If you intend to upgrade existing non-touch EPS320 Stimulator, you must purchase upgrade kit MP3113, which contains the required software version 3.20(SR3.1) upgrade.
2. Refer to Installation Manual (MP3069 or MP3156) supplied with this EPS320 Cardiac Stimulator for full installation instructions.
3. Connect Elo Touch screen mains power input to the Micropace MP3107 medical grade isolation transformer; connect video, serial and speaker cables to respective marked ports on the rear of the Bona Light PC (use COM2 or higher and the unmarked Audio Line Out jack).
4. Ensure all mains power supply is routed via medical grade Isolation Transformer such as Micropace PN MP3107.
5. Power up PC and follow the instructions to calibrate the ELO Touch screen. Make sure you calibrate it at the operator's sight level to the touch screen, using a stylus.
6. Verify correct installation by rebooting PC and using touch screen to select protocols and start / stop pacing.

Packing List

Touch Screen - MP3113-GE and MP3113-B		
Item	Part No.	Qty
ELO Touch Screen and accessories	MP3114	1
Touch Video Cable, 2 m	Part of	1
Touch screen Serial Cable, 2 m	Part of	1
Speaker Cable, 2 m	Part of	1
Universal Cloverleaf IEC mains power connector lead IEC320 – C5 to C14	MP3158	1
Touch Screen Instruction Manuals – this document & CD ROM	MP3160	1
Touch Screen Stylus and holder	MP3133	1
Touch Screen Quick User Guide	MP3136	1

See below for the contents of the ELO Touch Monitor.

